

NEW!

CoFlex[®]•AFD

ABSORBENT FOAM DRESSING

LATEX FREE

**An Easy,
Cost Effective
Wound Management
Solution!**

All-in-One Foam Pad Dressing and Cohesive Bandage


Features:



Benefits:

- High absorption and retention of exudate
 - Keeps wound site moist and clean
 - All-in-one dressing/cohesive bandage
 - Soft and comfortable CoFlex LF² base
 - Waterproof film layer over foam pad
 - Expands to conform to wound cavity
 - Provides controlled compression
 - Easy application
 - Non-adherent to wound bed
- Reduces risk of maceration
 - Promotes wound closure and natural healing
 - Quick - minimizes dressing time
 - Improves patient compliance – heals wounds faster
 - Prevents leakage, drying out and contamination
 - Manages exudate; reduces maceration around wound bed
 - Will not constrict over time
 - Can be applied by professional or patient
 - Protects delicate granulating tissue; avoids trauma during dressing changes

CoFlex AFD Construction

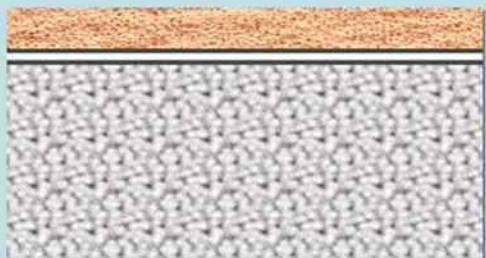

- ← CoFlex LF² Layer - Quick stick, cellular foam cohesive bandage is comfortable and lightweight for prolonged wear and increased patient compliance
- ← Waterproof Film - Prevents leakage, drying out and contamination
- ← Absorbent Foam - Foam pad draws in exudate, while keeping wound site moist and clean

TESTIMONIALS

“We were very happy with how easy the CoFlex AFD foam pad dressing was to apply. Patients we sent home with the product were able to change the dressings without difficulty and therefore more compliant. We used the CoFlex foam pad dressing on many types of wounds and were pleased with the healing we saw with them all.”

*Amy Mahoney
Boone Hospital, Wound Clinic
Columbia, MO*



“The feedback we received from the patients is great and I love the product. Very user friendly.”

*Gwen Mccomy
Clinical Director
Jefferson Memorial Hospital Wound Care Center*



Applications and Uses

- Venous Leg Ulcers
- Arterial Ulcers
- Pressure Ulcers
- Traumatic Wounds
- Diabetic Foot ulcers
- Non-healing Surgical Incisions
- Lower Extremity Wounds
- Skin Tears
- Burns
- Tracheotomies
- Donor Sites
- Chronic Wounds

General Wound Application



1) Unroll CoFlex AFD to expose the foam pad.



2) Place foam pad on treated, clean wound and wrap bandage at desired tension.



3) Continue wrapping and then seal end of bandage by pressing down and meshing layers together.

Venous Leg Ulcer Application



1) Unroll and place foam pad on treated, clean wound. Wrap once at desired tension. Hand-tear and seal bandage by pressing down and meshing together.



2) With foot flexed to 90°, continue application beginning with a circular winding of bandage at base of toes.



3) Proceed up and around the ankle, covering the Achilles tendon.



4) Wrap over the ankle and down to secure the heel.



5) Proceed up the leg in a spiral fashion, each turn covering 50% of the previous turn. End just below the knee.

CoFlex[®]·AFD

ABSORBENT FOAM DRESSING

ORDERING INFORMATION

U.S. Patent Nos. 5,762,623 & 6,156,424
 Patent Pending
 U.S. Patent No. 6,566,576 Dicon Technologies

Catalog#	Size	Rolls/Case
9720TN	2" x 2.5 yds.	16
9740TN	4" x 2.5 yds.	8
9840TN	4" x 7.5 yds.	6
9760TN	6" x 2.5 yds.	8



Sterile (S) = Sterile Product/Tan only.

9720(S)	2" x 2.5 yds.	16
9740(S)	4" x 2.5 yds.	8
9840(S)	4" x 7.5 yds.	6
9760(S)	6" x 2.5 yds.	8



Fenton Pharmaceuticals

4J Portman Mansions, Chiltern Street,
 London W1U 6NS

Telephone: 020 7224 1388 Fax: 020 7486 7258

mail@fent-pharm.co.uk

http://www.fentonpharmaceuticals.com