


INSTRUCTIONS FOR USE

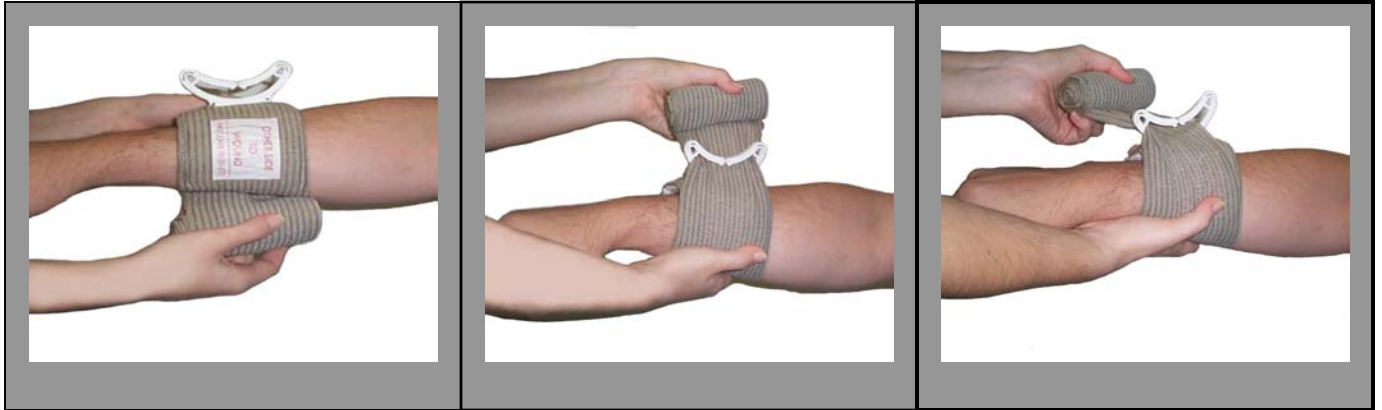

1 Place pad on wound & wrap the elastic bandage around limb or body part

2 Insert elastic bandage into pressure bar

3 Tighten elastic bandage



4 Pull back – forcing pressure bar down onto pad

5 Wrap elastic bandage tightly over pressure bar and wrap over all edges of pad

6 Secure hooking ends of closure bar into elastic bandage

THE EMERGENCY BANDAGE

Sole UK Distributor:
Fenton Pharmaceuticals Limited
4J Portman Mansions
Chiltern Street
London W1U 6NS
Telephone: 020 7224 1388
Fax: 020 7486 7258
Email: mail@fent-pharm.co.uk