



Intraosseous Infusion System

with depth-control mechanism for use with adults




1.

Clean.
Nettoyez.
Reinigen.
Limpiar la zona.
Pulire.




2.

Locate sternal notch.
Apply Patch.
Localisez l'encoche sternale.
Appliquez le patch.
Einkerbung am Manubrium
lokalisieren. Pflaster aufbringen.
Localizar la horquilla esternal.
Colocar el parche.
Individuare l'incisura sternale.
Applicare il Patch.

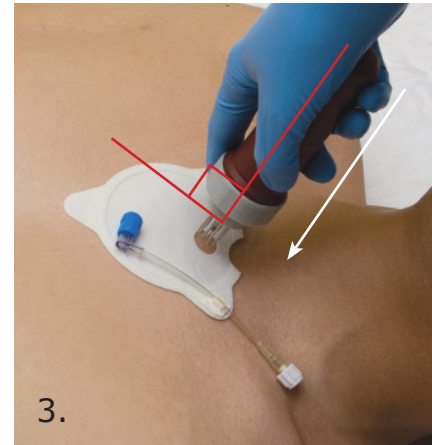

3.

Insert Introducer.
Insérer l'introducteur.
Einführhilfe einführen.
Insertar el introductor.
Inserire l'introdotto.

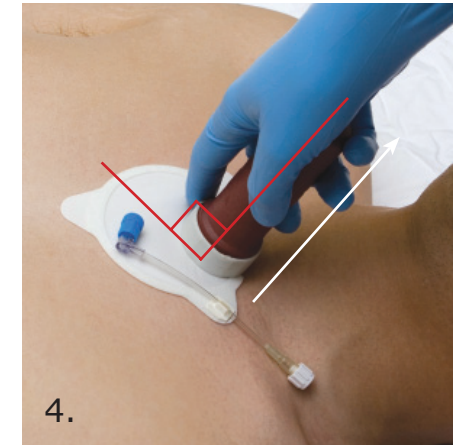

4.

Withdraw Introducer.
Retirez l'introducteur.
Einführhilfe zurückziehen.
Retirar el introductor.
Estrarre l'introdotto.



5.

Connect tubing.
Connecter la tubulure.
Schlauch anschließen.
Conectar el tubo.
Collegare la tubazione.



6.

Place Dome.
Placez le dôme.
Schutzabdeckung aufsetzen.
Colocar el domo.
Collocare la cupola.

PYNG MEDICAL

#7-13511 Crestwood Place
Richmond BC V6V 2E9 Canada
1-800-349-7964
info@pyng.com
www.pyng.com

Authorized Representative
(Regulatory Affairs)
EMERGO EUROPE,
Molenstraat 15
2513 GH The Hague,
The Netherlands
(+31) 70 345 8570



Remove Infusion Tube.
Retirer le tube de perfusion.
Infusionsschlauch entfernen.
Retire el tubo de infusión.
Rimuovere il tubo di infusione.